Liners & Sleeves

Instructions for Use - *for patients*

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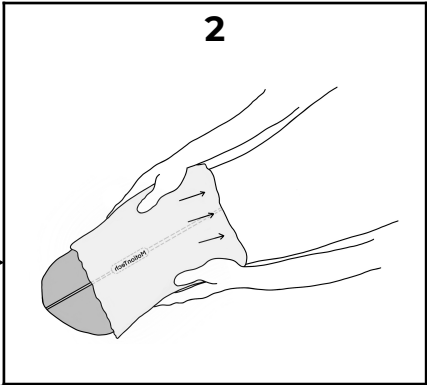
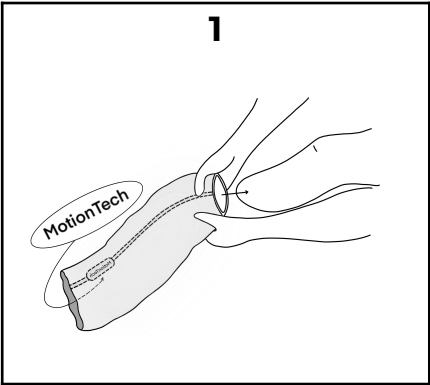
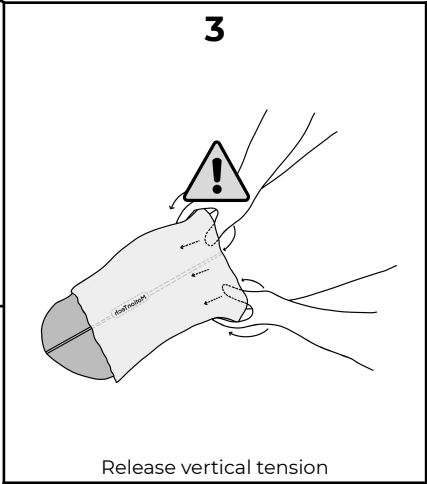
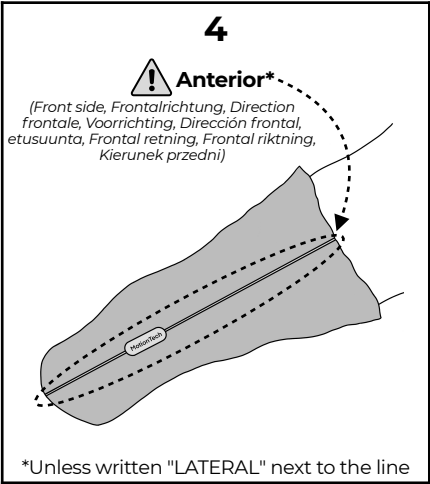
NOTES

Introduction

Dear user, congratulations on starting to use Your® Liner and/or Sleeve. We hope you will enjoy wearing it as much as we enjoyed crafting it-for you!

Your® product was passionately crafted by our prosthetic experts. We should be wearing it with a prosthetic technician, taking into account your inputs, to come up with the best design for you.

In order to enjoy the optimal performance and comfort while using Your® product, it is important to apply the following instructions.

General Disclaimer

MotionTech retains all intellectual property rights reflected in its physical products, regardless of the transfer of the physical products to another party or parties.

For safety reasons, Your® Liner can only be ordered by a certified prosthetic technician.

*The word «product» used in this document applies to both Your® Liners and Your® Sleeves.

Intended use of Your® Liners

A Your® Liner is a protective cover designed to be worn over a residual limb. A Your® Liner is intended to be in direct contact with the intact skin of the stump it was designed for. A Your® Liner is intended to provide a slight compression onto its user's residual limb.

A Your® Liner is intended to be used in conjunction with a prosthetic socket. Its function is to provide cover for the residual limb, and mitigate the load transferred from the residual limb to the prosthesis over the various tissues.

A Your® Liner is intended to be worn for a maximum of 8h in a row. It should not be doffed and cleaned according to cleaning instructions provided in the present booklet.

Intended use of Your® Sleeves

A Your® Sleeve is a prosthetic interface with suspension properties designed to be worn over a residual limb. A Your® Sleeve is intended to be used in conjunction with a prosthetic socket. The proximal end of a Your® Sleeve is intended to be in direct contact with the intact skin of the stump it was designed for, while distal end is intended to be worn over the prosthetic socket.

The function of a Your® Sleeve is to ensure adequate suspension of the prosthesis to the residual limb.

A Your® Sleeve is intended to be worn for a maximum of 8h in a row. It should then be doffed and cleaned according to cleaning instructions provided in the present booklet.

1. Donning

Your® Liner/Sleeve was designed specifically for your limb. It presents a unique geometry and is intended to be carefully positioned and aligned when used in order to offer the comfort that we want you to enjoy.

It is therefore **crucial** that you take the time to position it correctly when donning Your® Liner/Sleeve. When donning Your® Liner/Sleeve, turn it inside out so that the fabric cover is on the inside. Your limb should only be in contact with the bare silicone surface, never with the fabric-covered one.

Locate the alignment line printed on the textile, and orient it so that it will be on its intended direction once the liner is donned on the stump: Anterior (if written "Posterior" on other side) or Lateral (if written "Medial" on other side). See image 1.

NOTES

- Before donning, check that the inside of the product is clean, dry and that there are no external objects, substances or items that can cause skin irritation.
- Before donning, check that the product is not hot, to avoid burning your skin while donning.
- Never don the product with the fabric cover in direct contact with your skin as this could lead to skin irritations.
- Be careful not to damage the product with your fingernails.
- Do not pull the product up from the proximal end; as it may stretch your skin and cause discomfort, blisters or rashes, or lead to loss of suspension
- Once donned on, make sure that there are no wrinkles or air pockets between the product and your limb. These could cause skin irritations or lead to loss of suspension

1.1. Donning a Your® Liner

Position the end of the liner against the end of your limb, and start rolling it up onto your limb, paying attention not to leave air pockets between your limb and the liner. See image 2.

After rolling up a segment of ~5cm, **release the vertical tension** which may have accumulated in the liner by sliding your thumbs between the liner and your skin to "push down" the internal surface of the liner to a relaxed state. See image 3.

Repeat the "rolling up" and "releasing vertical tension / pushing down" steps until the liner is fully donned onto your limb. See image 4.

1.2. Donning a Your® Sleeve

Position the distal end of the sleeve on the prosthetic socket. Once the liner is donned, insert your limb in the socket and roll the sleeve up onto your limb, paying attention not to leave air pockets between the sleeve and the socket / your limb.

Be careful not to don the sleeve with the proximal and distal ends reversed, as this could lead to excessive compression on your limb, or induce failure of the suspension.

1.3. Landmarks to check the donning

In order to make sure that you aligned Your® Liner/Sleeve correctly, if applicable, check that the thicker parts of Your® Liner/Sleeve are correctly covering the anatomical structures they are supposed to, as your prosthetist indicated when she provided you with the Your® Liner/Sleeve. The alignment line printed on the textile should be positioned towards the front (if written "Posterior" on the other side of the product) or to the lateral side (if written "Medial" on the other side of the product). If the line is not correctly positioned, start over.

For a Your® Sleeve, the horizontal line which extends from both sides of the vertical line must be positioned at the level of the middle of the articulation (knee, elbow, etc.). To make sure Your® Sleeve is oriented in the right direction,

- The first time you put on Your® Liner/Sleeve, your prosthetist should check that it is conform to his/her design indications. If applicable, she should explain which zones are covered by extra cushioning and why, in order to help you align Your® Liner/Sleeve.

Your® prosthetist must also ensure that Your® Sleeve provides a safe and adequate suspension to the prosthesis before letting you go with it. Moreover, before each use, ensure the suspension of the prosthesis is safe and adequate by taking a few steps in a safe area, to guarantee that Your® Sleeve is correctly positioned and is functioning correctly.

Suspension

2.1. Your® Cushion Liners

A Your® Cushion Liner does not play a role in the suspension of the prosthesis. A Your® Cushion Liner is designed to work with a suspension sleeve. Even if your prosthetic technician provided you with a socket that also adds some anatomical suspension, DO NOT use your liner with the rest of your prosthesis without the suspension sleeve. This may result in slippage of the prosthesis on the liner.

2.2. Your® Pin-lock Liners

Your® Pin-lock Liners are based on a pin-lock suspension mechanism. A Your® Pin-lock Liner is intended to be used with a plunger pin and a locking mechanism. A locking mechanism therefore needs to be integrated in the prosthesis and a plunger pin compatible with the chosen locking mechanism needs to be fastened to the distal umbrella of the Your® Pin-lock Liner by the prosthetic technician, prior to provision of the Your® Pin-lock Liner to its intended patient. Consult your prosthetic technician for additional instructions on how to properly use this type of liners.

Every effort has been implemented to ensure that the suspension strength of Your® Pin-lock Liners is as high as possible. However, as Your® Pin-lock Liners are tailor-made, classical strength tests are not sufficient to ensure a flawless mechanical strength for each designed product. If any sign of excessive movement or product wear is detected on or around the locking part of the liner at any time, stop using the device immediately and consult your physician or prosthetic technician.

Do not use Your® Femoral Pin-Lock Liner without a supplementary suspension method if you present a short residual limb. This condition may lead to loss of suspension.

2.3. Your® Suction Liners

Your® Suction Liners are based on a vacuum suspension system. A Your® Suction Liner is intended to be used with a dedicated prosthetic socket equipped with a standard expulsion valve. A Your® Suction Liner must also be used with a supplementary suspension system independent from the liner (for example a suspension sleeve, a Silesian belt or a Pelvic band), in case the patient presents a short residual limb, substantial daily volume fluctuations, or if sufficient suspension can not be achieved without such supplementary suspension system. Consult your prosthetic technician for additional instructions on how to properly use this type of liners.

Extensive efforts have been implemented to ensure that the suspension strength of Your® Suction Liners is as high as possible. However, as Your® Suction Liners are tailor-made, classical strength tests are not sufficient to ensure a flawless mechanical strength for each designed product. If any sign of excessive movement or product wear is detected on or around the silicone rings of the liner at any time, stop using the device immediately consult your physician or prosthetic technician.

Do not use a Your® Suction Liner without supplementary suspension method if you present a short residual limb or substantial residual limb volume fluctuations during the day. These two conditions may lead to loss of suspension.

Doffing

When doffing the product, roll it off from the proximal end. Make sure you do not stretch, pinch or pull on the product to avoid causing any damage to the product.

NOTE

Do not try to pull your product by its distal end, as this will create vacuum between your product and your limb, which may induce various skin problems.

Product cleaning and care

Adequate hygiene is essential to limit risks of irritation, and other skin problems.

4.1. Daily product cleaning

Your® Liner/Sleeve should be cleaned thoroughly at the end of each day where you wore it. Use lukewarm tap water and pH neutral soap that does not irritate the skin, and make sure you meticulously wash the interior surface, that comes directly in contact with your skin. Apply the soap with a clean soft cloth or sponge. Make sure that you rinse all the soap residue from the product with water, then dry the product with a clean lint-free cloth. Place the product in its normal position, with the fabric cover side out, on a drying stand. Do not let Your® Liner/Sleeve dry inside out.

NOTES

- Do not scrub Your® Liner/Sleeve, as it may roughen its surface, and cause various skin problems.
- Fabric softeners, bleaches, and other products or cleaning solutions may damage Your® Liner/Sleeve and should not be used.
- Do not clean Your® Liner/Sleeve with products that can cause skin irritation. Residues may remain and harm your skin when using Your® Liner/Sleeve.
- Do not try to dry Your® Liner/Sleeve by using a microwave, oven, dryer, or any electric or heating apparatus, as it may damage it. You might also burn yourself when donning a heated Your® Liner/Sleeve. Dry Your® Liner/Sleeve only with a clean lint free cloth.

4.2. Weekly disinfection
Place a small amount of ethyl or isopropyl alcohol on a soft, clean lint-free cloth. Softly rub the inside of Your® Liner/Sleeve with the cloth for two minutes. Thoroughly rinse off any excess alcohol with water.
Place the product in its normal position, with the fabric cover side out, on a drying stand. Do not let Your® Liner/Sleeve dry inside out.

4.3. Daily inspection
After each day where you used Your® Liner/Sleeve, you should inspect it for indications of damage or wear. These may be signs of changes in the fit of your prosthetic socket. Any damage may reduce the efficiency of the product and result in potential loss of suspension.

 NOTES

- If you use a Your® Pin-lock Liner, you should be especially cautious to observe if damage is present on or around the locking part of the liner. In case of doubt, stop using Your® Liner immediately and report the defects to your prosthetic technician. Failure of the locking mechanism may result in loss of suspension.*
- If you use a Your® Suction Liner, you should be especially cautious to observe if damage is present on or around the silicone rings of the liner, and report it immediately to your prosthetic technician. Failure of the silicone rings may result in loss of suspension.*
- If you use a Your® Sleeve, you should be especially cautious to look for damage which may generate air leaks (e.g. holes in the product) which could lead to loss of suspension. In case of doubt, stop using Your® Sleeve immediately and report the defects to your prosthetist.*

In order to preserve the warranty and ensure your safety while using your prosthesis, contact your prosthetic technician immediately if you notice damage or wear on Your® Liner/Sleeve. The same shall be done if you notice the seam starts unraveling or if the fabric starts to come off the elasticomeric body of Your® Liner/Sleeve.

4.4. Storage

If you are not wearing Your® Liner/Sleeve, store it with the fabric cover on the outside, in a cool, dry place. If you do not use it for an extended amount of time, make sure Your® Liner/Sleeve is dry, and cover it with a clean plastic bag. Do not store Your® Liner/Sleeve under direct sun exposure, as heating and cooling cycles may deteriorate the mechanical performance of Your® Liner/Sleeve.

 NOTE

Take care not to expose Your® Liner/Sleeve to glass or carbon fibres or other foreign particles. Such substances can become embedded in the silicone causing skin problems. Washing alone may not be sufficient to eliminate the problem. Return Your® Liner/Sleeve to your prosthetic technician if it is inadvertently exposed to any foreign substances or chemicals.

5. Skin care

The daily use of a prosthesis inherently exposes the skin of your limb to various mechanical stresses. Daily cleaning of your limb is therefore essential. We recommend the use of a mild liquid, pH balanced, 100% fragrance and dye-free soap.
If you notice dry skin, apply a pH balanced, 100% fragrance- and dye-free lotion to nourish and soften the skin. Do not apply lotion to your limb immediately before donning Your® Liner/Sleeve.

 NOTE

Many common household or bath products, including soaps, deodorants, perfumes, aerosol or alcohol sprays or abrasive cleaners may cause or contribute to skin irritation. Be cautious which products you use!

6. Warranty

6.1. Product warranty

MotionTech warrants that each delivered product will, at the time of delivery, be of workmanlike quality and substantially free of defects. MotionTech makes no other warranty, implied or expressed, and makes no warranty of merchantability or fitness for a particular purpose. This warranty shall terminate immediately upon any action to combine our products with other materials, or in any manner that change the nature of our products. MotionTech's liability shall not exceed the purchase price of the product. MotionTech is not responsible for any indirect, incidental or consequential damage. There is no warranty, expressed or implied, for damage caused by abuse, mishandling or accident.
MotionTech offers a warranty on each product against defects in material and workmanship, for a period of 6 months for a Your® Liner and 3 months for a Your® Sleeve counting from the date of MotionTech's invoice. MotionTech doesn't provide a warranty against normal wear & tear occurring under normal use conditions. For patients with a mobility level of K3 or K4 who are fitted with a Your® Liner which contains Relax silicone, the warranty is reduced to 3 months. If a replacement product is provided to a customer, the warranty period will be the number of days remaining on the original warranty. The durability of Your® Liners & Sleeves may vary based on specific design parameters (e.g., silicone type, thickness). MotionTech reserves the right to adjust the applicable warranty period on a case-by-case basis if scientific design features are known to reduce product durability. Such adjustments shall take effect upon written notice, including via email.
This warranty shall be in lieu of all others, whether expressed or implied by law. There is no warranty, expressed or implied, for damage caused by abuse, mishandling or accident. The warranty is void if the Instructions for Use are not followed.

6.2. Service warranty

Your® Liners/Sleeves are manufactured based on specifications provided by a prosthetic technician. If a buyer is not satisfied with the product, it may be returned within 30 days of invoice for alteration, replacement, or credit at the sole discretion of MotionTech. Credit will only be applied if it is determined the product is not appropriate for the intended patient. Returns will not be accepted for customer errors.

6.3. Complaints, return requests, non-conformities, incidents

In case you want to file a complaint, report a non-conformity or an incident, MotionTech provides you with several channels:

• Email: You can send your request at support@motiontech.ch
• Ordering Platform: Your prosthetic technician can file complaints, return requests, report non-conformities or incidents online on MotionTech's Ordering Platform: <https://motiontech.ch/yourlinerorderingplatform>

7. Warnings & Precautions

 Your® Liner/Sleeve should not be worn for more than **8h** in a row. After that time, Your® Liner/Sleeve should be doffed and cleaned as aforementioned.
 Your® Liner/Sleeve has been designed specifically for you. It is intended only for your usage, not for other patients. Use of Your® Liner/Sleeve by multiple patients could lead to cross contamination between patients. Other patients may also cause themselves injuries if using Your® Liner/Sleeve, due to unadapted shape.

 Your® Liner/Sleeve is intended to be in contact with and only with the skin of the stump it was designed for. If you have a bilateral amputation do not try Your® Liner/Sleeve on the limb it was not intended for.

 The skin on which Your® Liner/Sleeve is placed needs to be intact. If any wounds, rashes, redness, eruptions, blisters or any other injuries are present on your limb, do not wear Your® Liner/Sleeve until the injuries are healed. Consult your prosthetic technician or physician if you have persistent skin injuries that may require professional medical attention.

 To avoid danger of suffocation, keep Your® Liner/Sleeve away from infants and children, and do not try donning Your® Liner/Sleeve on your head in any case.

 Your® Liner/Sleeve can melt or burn if exposed to high temperatures or flame. Do not expose Your® Liner/Sleeve to these conditions.

 Your® Liner/Sleeve shall be used only for the purpose and in the manner described in this document. MotionTech declines all responsibility if any harm, material or physical, occurs while using its products for another purpose.

 Your® Liner/Sleeve are made of silicone materials. If you are allergic to silicone, do not use these products.

 Do not use a Your® Sleeve if you notice there is a hole in the silicone body, as this may result in loss of suspension leading to fall.

 Your® Liners/Sleeves are tailor made based on specific design prescriptions. Depending on this design prescription, on usage conditions and on your activity level, their lifetime can differ substantially.

 After using Your® Liners/Sleeves for 6 months, you should contact your prosthetic technician for replacement, even if it seems to be in good condition. Wearing a deteriorated product may put you at risk of rashes and other skin problems.

 When disposing of a worn out Your® Liner/Sleeve or providing it to anyone, please thoroughly implement the cleaning and disinfections procedures described in section 4. Once free of potential traces of blood, excretions or secretions, Your® Liner/Sleeve can be disposed of as normal domestic waste, in accordance with your local waste disposal regulations.

 Should any serious incident involving a Your® Liner/Sleeve occur, please report it immediately to MotionTech and to your national competent authority for medical devices.

 Your® Pin-Lock Liners include metal parts. Do not wear Your® Pin-Lock Liner if you engage in a therapeutic procedure which involves magnetic fields and/or radiation.

 Your® Liner/Sleeve has been designed specifically for the shape of your limb. If your residual limb has undergone significant volume changes since Your® Liner/Sleeve was delivered to you, you should stop using it. Using an ill-fitted Your® Liner/Sleeve may put you at risk of injuries, or of your limb slipping out of the Your® Liner/Sleeve.

 When you engage in an activity that could result in excessive perspiration make sure to periodically remove Your® Liner/Sleeve to dry it off. Similarly, make sure both your stump and Your® Liner/Sleeve are dry before donning. Wearing a wet Your® Liner/Sleeve involved in the suspension of the prosthesis (i.e. Pin-lock, Suction liners, and sleeves) exposes you to the risk of falling.

 We do not recommend using Your® Liner/Sleeve when engaging in extreme activities (e.g. skydiving) as your prosthesis might slip off.

 If using it underwater, water might come between your limb and Your® Liner/Sleeve. The Your® Liner/Sleeve risks slipping off your limb and you risk losing your prosthesis. If using Your® Sleeve underwater, we recommend that you always check first in a safe place that the suspension stays safe and strong once submerged, before risking going in an area where you might fall or loose the prosthesis.

 After using Your® Liner/Sleeve in a wet environment or under water, make sure to dry it off right away. Using a wet Your® Liner / Sleeve may put you at risk of falling.

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